


Designed for Quality Pressure Relief

- ◆ Supine offloading (bilateral placement).
- ◆ Settles persistent patient shifting.
- ◆ Stable during continuous rotation turning.
- ◆ Pressure relief for Hemodynamically unstable.

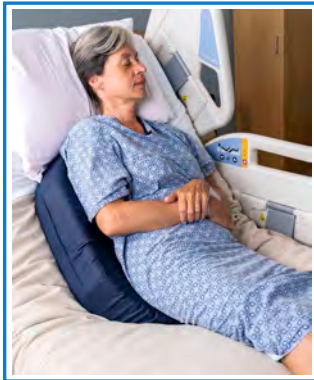

- » Semi-Fowler: Flexes to remain in place in semi-fowler



- » Conforming Shape
- » Consistent Pressure Relief
- » Comfortable Stability

- » Even lift from scapula & buttocks
- » Circumvents sacrococcygeal area



Bilaterally Equal Angle Sides



- » Precise 30o angle
- » Uniformly distributed support
- » Musculoskeletal alignment



- » Supine Pressure Relief: The only 30-degree lateral positioning wedge that can relieve pressure in the supine position. (using 2)

- ◆ Medical Grade Polyurethane coated knit cover
- ◆ High vapor air exchange
- ◆ Low friction – won't stick to skin
- ◆ Moisture Resistant
- ◆ Antimicrobial treated
- ◆ Tolerates all disinfectant wipes, soap & water

The Bedsore Rescue™ Positioning Wedge Cushion Specifications

	Standard Size 27" L x 12" W Wedge 5" at back to 2" at front tips	Large Size 28" L x 12" W Wedge 6" at back to 2.5" at front tips	Bariatric Size 29" L x 18" W Wedge 8" at back to 3" at front tips
Persons Weight	90 to 180bs	181-350 pounds	> 351 pounds
Persons Height	5.0" to 6'.5"	5.0" to 6'.5"	5.0" to 6'.5"
Material	<p>Inner Core: High resolution medium density polyurethane foam</p> <p>Support layer: Pressure distributive high density memory foam</p> <p>Latex free - Hypoallergenic</p>		
Cover	<ul style="list-style-type: none"> Home use model: 100% Chemflex II ® Polyester knit "gurney pad" fabric Non-skid Model: Top: Chemflex II ® Polyester knit "gurney pad" fabric Bottom: roughened rubber Non-Skid designed for use on low air loss & alternating air mattress surfaces <p>ChemTek II Fabric Specifications:</p> <ul style="list-style-type: none"> Moisture tolerance: Proof Resistant to 150lbs/per sq. inch Moisture vapor exchange: (breathability) 194 G/m2/24hr (soft feel, skin safe) Treatment: Antimicrobial treated, hypoallergenic Construction: Fine stitch fixed. Contains no metals, no zippers 		
Disinfection	<p>This product is reusable. Antimicrobially treated to inhibit colonization. Wipe or rinse to clean and disinfect between uses. Compatible with bleach, hydrogen peroxide, chlorhexidine based disinfectant wipes or soap & water. Do not machine wash. Do not submerge. Discard if cover damaged or contamination is suspected. We recommend discarding after use in COVID-19 isolation.</p>		
<p>For best results, place cushion under incontinence pads and over slide/boost sheets. Please see "How to Place" and "How to Clean" instructions for details</p>			
Ordering	<p>www.jewellnursingsolutions.com, Or call (650) 294-8557 Bulk pricing available</p>		
Guarantee & Warranty	<p>6 months warranty for materials & construction under normal wear and tear (Excludes damage from to sharp objects, corrosive fluids, submersion in fluids (laundering, tubs), and exposure to temperatures >200 degrees). 10-day Satisfaction guarantee or your money back when purchasing 2 or less cushions.</p>		

Meets flammability requirements of electronic & appliance repair, home furnishing and thermal insulation technical bulletin 117-2013
Contains NO added flame-retardant chemicals

Jewell Nursing Solutions is a nurse owned and operated company dedicated to bringing effective pressure injury prevention products and services to the entire caregiving community. Bedsore Rescue™ Cushion (formally The BackBone™) is proudly designed, patented, and trademarked by Jewell Nursing Solutions. A certified woman owned business celebrating 10 years of helping people stop Pressure Injuries. Made in the USA by certified manufacturer reg #CA39930 (IL). Distributed by Jewell Nursing Solutions